

- Small and lightweight one-piece design
- For newborns & low-birth weight babies
- Ready to use – for distribution in hospitals only



°BPA/BPS Free: All MAM products are made from materials free of BPA & BPS




Instructions for use:

Please read these instructions carefully before first use!

- Germ-reduced through gamma irradiation and therefore ready for initial use, provided the packaging is intact.
- If packaging is damaged, clean and sterilize before use. Allow to cool.
- Clean before each further use. Wash with mild soapy water and then rinse with clear water.
- During cleaning a small amount of water may enter the nipple. This can be expelled by squeezing the nipple. This is to ensure hygiene.
- Sterilizing methods: steam sterilizing (electric or microwave), immersing in a liquid sterilizing solution, all according to manufacturer's instructions, or boiling in water for at least 5 minutes.
- Autoclavable up to 134°C/273°F for a maximum of 18 minutes (max. 20 times).
- After cleaning or autoclaving expel remaining water by squeezing the nipple with sterilized nipple tongs.
- Do not use aggressive cleaning agents.
- Never dip the nipple in sweet substances or medication. This can cause tooth decay.
- To ensure safety and hygiene replace the pacifier every 1–2 months.
- Do not worry if the pacifier becomes lodged in the mouth. It can not be swallowed and is designed to cope with such an event. Remove as gently as possible.
- Do not worry if the child's finger gets stuck in any opening of the pacifier. The pacifier is made from soft silicone and therefore does not restrict blood circulation. Remove as gently as possible.
- MAM recommends giving up pacifier usage at 3 years of age.

⚠ For your child's safety WARNING!

Inspect carefully before each use. Pull the pacifier in all directions. Throw away at the first signs of damage or weakness. Only use dedicated pacifier holders tested to EN 12586. Never attach other ribbons or cords to a pacifier, your child may be strangled by them.

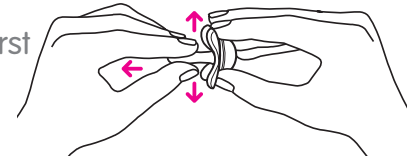

Do not leave a pacifier in direct sunlight or near a source of heat or leave in sterilizing solution for longer than recommended, as this may weaken the nipple. To avoid danger of suffocation, keep the plastic bag away from babies and children. Do not use the bag in cribs, beds, carriages or playpens. The bag is not a toy. Do not tie pacifier around child's neck as it presents a strangulation danger.

How to store this product

The pacifier has to be stored at room temperature between + 5°C and + 30°C / + 41°F and + 86°F.

Protect from direct sunlight.

Expiration date is on the packaging.

Lot number is on the packaging.

MAM Easy Start™ Anti-Colic Bottle:



DEVELOPED WITH MEDICAL EXPERTS

Teamwork with medical experts for maximum safety

Only after approval by medical experts is MAM innovation ready for baby life.




Scientific Panel of
BAMED AG
www.icmrs.org

Nipple with SkinSoft™ Silicone surface feels familiar to babies.

MAM Pacifiers are the result of our close collaboration with pediatricians, dentists and orthodontists. Comfortable for babies, reassuring for parents and with a familiar nipple feeling. The special nipple and shield design ensure a perfect fit and skin-friendly shape.

Available in different sizes – to ensure safety and comfort for all ages.

**Special size for MAM Comfort™

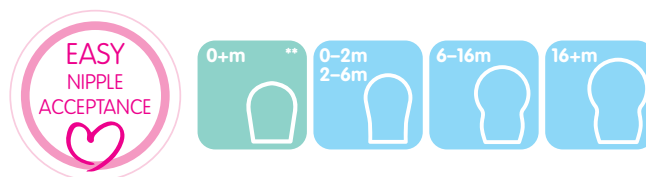

MAM Premies & MAM Comfort™

Because every baby is special: MAM has developed a pacifier line especially for the needs of premature and low-birth-weight babies.



The product meets the requirements of US Standard 16 CFR 1511 and EN 1400, except paragraph 13.3.5.

MAM International AG, Sihleggstrasse 15,
8832 Wollerau, Switzerland.
Made in Austria.

For more information please contact:
MAM USA Corporation
2700 Westchester Avenue
Suite 315
Purchase, NY 10577
866-949-1174

More MAM products on mambaby.com
Please keep instructions for future use.