

INSTRUCTIONS

- Pacifier for newborn infants
- Ready to use – for distribution in hospitals only
- Latex-free
- Lightweight one-piece construction

1,400 –
6,500 g




Product complies with
CFR 1511 and EN 1400

BPA°/BPS Free: All MAM products
are made from materials free of
BPA and BPS

⚠ WARNING: Do not tie pacifier around child's neck as it presents a strangulation danger. Inspect carefully before each use. Pull the pacifier in all directions. Throw away at the first signs of damage or weakness.

⚠ WARNING: To avoid danger of suffocation, keep this plastic bag away from babies and children. Do not use this bag in cribs, beds, carriages or playpens. This bag is not a toy.

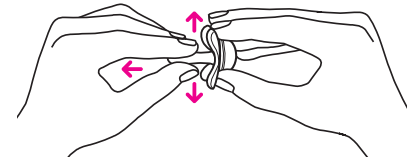

Germ-reduced through gamma irradiation and ready for initial use if package is not opened or damaged. If packaging is damaged, clean and autoclave before use. Allow to cool. Clean after each use. Wash with mild soapy water and then rinse with clear water. Do not use aggressive cleaning agents. Autoclavable up to 134°C/273°F. Allow to stand for a further 5 minutes before giving it to baby.

MAM International AG, Sihleggstrasse 15,
8832 Wollerau, Switzerland.

Made in Austria.

For more information please contact:

MAM USA Corporation
2700 Westchester Avenue

Suite 315

Purchase, NY 10577

866-949-1174

Ready to use

Not for resale