

PREEMIES SIZE 1 & 2

MAM

- Pacifier for premature & low-birth-weight babies
- Ready to use – for distribution & use in hospitals only
- Latex-free

SIZE 1
<
1,000 g

SIZE 2
900 –
1,500 g

BPA°
BPS
free

Skin
Soft™
Silicone

Instructions for use

Please read these instructions carefully before first use! These instructions only apply to MAM Premies size 1 and size 2. Separate instructions are available for the MAM Comfort™.

CE Medical Device Class 1

BPA°/BPS Free: All MAM products are made from materials free of BPA & BPS

The benefits of pacifiers

MAM Premie pacifiers are intended to be used for premature babies up to a maximum body weight of 1,500 g under medical supervision to compensate for the typically present suck and swallow deficiency. Pacifiers help to satisfy the natural sucking urge of babies. Babies can benefit from the pain-relieving effect of pacifiers during medical interventions. Furthermore, pacifiers help with the switch from gavage feeding to breastfeeding or bottle feeding as it trains the oral musculature. The pacifier is one of the many preventive strategies against sudden infant death syndrome (SIDS).



When to use this product

The MAM Premies are intended for premature babies up to 1,500 g under medical supervision.

MAM Premie size 1- for babies with a weight below 1,000 g

MAM Premie size 2- for babies with a weight between 900 g and 1,500 g

The pacifier is only intended for a premature baby in the respective weight range (see print on bag) and may only be used during hospital stay.

How to clean & use this product

MAM Premies are germ-reduced through gamma irradiation and ready for initial use if package is not opened or damaged. If packaging is damaged, clean and autoclave before use. Allow to cool. Clean after each use. Wash with mild soapy water and then rinse with clear water. Do not use aggressive cleaning agents. Autoclavable up to 134°C/273°F for a maximum of 18 minutes (max. 20 times). Allow to stand for at least 5 minutes before giving to baby. After cleaning or autoclaving expel remaining water by squeezing the nipple with sterilized nipple tongs. Replace the pacifier at least every month. Only use for one baby. Sizes and weights are general guidelines. Individual adaption and usage with oral & nasal tubing: the product can be tailored by cutting with scissors or a scalpel (see fig. 1 and fig. 2). Dispose of all small parts immediately after cutting to prevent choking hazards.


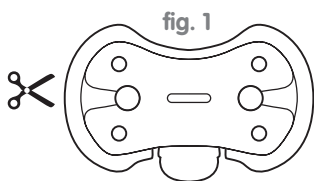


fig. 1

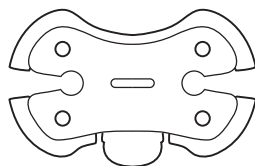

Minimum version
Premie Size 1


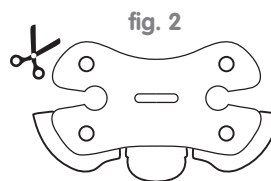


fig. 2

Premie Size 2 only!

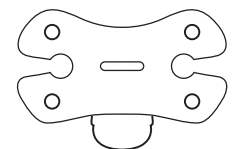

Minimum version
Premie Size 2

When this product may not be used

The MAM Premies size 1 & 2 may only be used during hospital stay under medical supervision.

Therefore, parents must NEVER be allowed to take the product home.


MAM Preemie size 1 must not be used for babies above 1,000 g.
MAM Preemie size 2 must not be used for babies below 900 g or above 1,500 g.
The size of the pacifier is embossed on the nipple.

⚠ WARNING! For your child's safety and health.

WARNING: This pacifier is for a premature infant. Use of this product beyond the recommended age for the baby could pose a choking risk.

Do not use this product when the infant reaches the developmental age of 27 Weeks (<1,000 g) in case of Preemie 1 or 30 weeks (<1,500 g) in case of Preemie 2. Please consult with your doctor.

Inspect carefully before each use. Pull the pacifier in all directions. Throw away at the first signs of damage or weakness. Do not leave the pacifier in direct sunlight or near a source of heat (except during cleaning) or leave in cleansing solution for longer than recommended, as this may weaken the nipple. To avoid danger of suffocation, keep the plastic bag away from babies and children. Do not use the bag in cribs, beds, carriages or playpens. The bag is not a toy. Do not tie pacifier around child's neck as it presents a strangulation danger. The product does not meet the safety standards which are applied to normal pacifiers, namely EN 1400 and 16 CFR 1511. It poses a choking hazard. Therefore, it must only be used under medical supervision. Parents must never be allowed to take the product home.

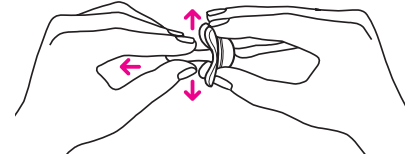

Composition of this product

The MAM Preemies are made of 100% odorless silicone.

How to store and dispose of this product

The pacifier has to be stored at room temperature between + 5°C and + 30°C / + 41°F and + 86°F.

Protect from direct sunlight.


Best used by date is on the packaging. Replace the pacifier at least every month. Dispose of this product in general hospital waste.

Lot number is on the packaging.

Date of information: 10/2018 – V.1.0

MAM Premies & MAM Comfort™

Every baby is special: MAM has developed a pacifier line especially for the needs of premature and low birth weight babies.

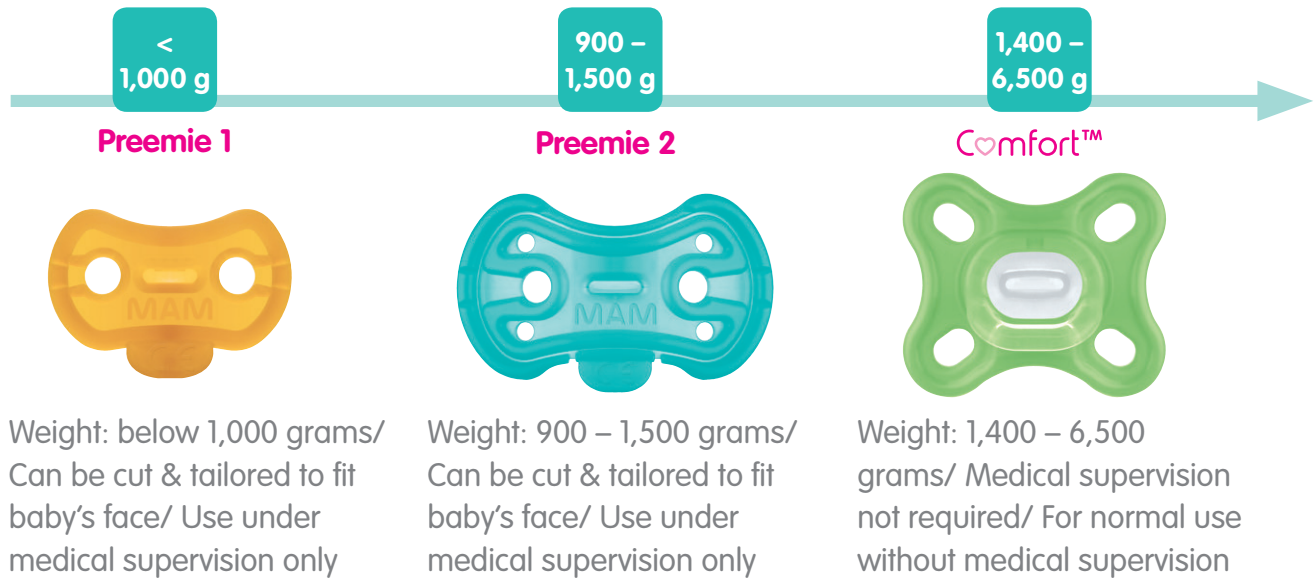

Patent Number: US 7789894

MAM International AG, Sihleggstrasse 15,
8832 Wollerau, Switzerland.

Made in Austria.

For more information please contact:

MAM USA Corporation
2700 Westchester Avenue
Suite 315

Purchase, NY 10577

914-468-4048

NICUcustomerservice@mambaby.com

More MAM products on mambaby.com

Please keep instructions for future use.