

INSTRUCTIONS

- Pacifier for premature & low-birth-weight babies
- Ready to use – for distribution & use in hospitals only
- Latex-free
- Lightweight one-piece construction

900 –
1,500 g



Medical Device





°BPA/BPS Free: All MAM products are made from materials free of BPA and BPS

⚠ For your child's safety WARNING!

Inspect carefully before each use. Pull the pacifier in all directions. Throw away at the first signs of damage or weakness. To avoid danger of suffocation, keep the plastic bag away from babies and children. Do not use the bag in cribs, beds, carriages or playpens. The bag is not a toy. Do not tie pacifier around child's neck as it presents a strangulation danger. The product does not meet the safety standards which are applied to normal pacifiers, namely EN 1400 and 16 CFR 1511. It poses a choking hazard. Therefore, it must only be used under medical supervision. Parents must never be allowed to take the product home.

Read Instructions for use:

Germ reduced through gamma irradiation and ready for initial use. Follow instructions for use on how to clean and autoclave before use, individual adaption (fig.1) and usage with oral & nasal tubing.

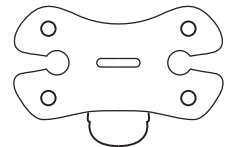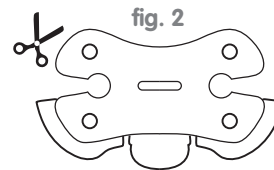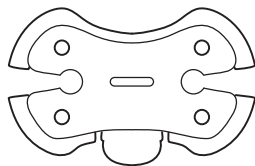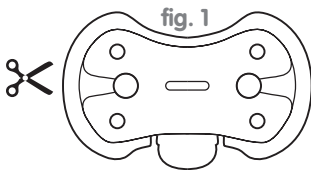

Minimum Version

MAM International AG, Sihleggstrasse 15,
8832 Wollerau, Switzerland.

Made in Austria.

For more information please contact:

MAM USA Corporation
2700 Westchester Avenue

Suite 315

Purchase, NY 10577

914-468-4048

NICUcustomerservice@mambaby.com

Ready to use

Not for general sale