



Nasal Aspirator

Online Shop

EN

Nasal Aspirator

Instructions for use

Wash before first use. Compress aspirator to eliminate air. Gently place tip into one nostril while keeping the other closed. Slowly release pressure in the aspirator and the suction will draw mucus from baby's nasal passage. Repeat as necessary. Clean immediately after use. Suitable for sterilizers. Top rack dishwasher safe.

FOR THE SAFETY OF YOUR CHILD –

WARNING!

NEVER compress aspirator while inside baby's nose. NEVER use in baby's mouth or ears. This product is not a toy! Contains small parts: to avoid a risk of injury, store out of reach of children. Inspect before and after each use. Throw away at first sign of damage or weakness. Please retain packaging for future reference. Not recommended for use with breastmilk.