
MAM Preemie soother/ pacifier for premature babies, Chupeta MAM Preemie para bebés prematuros, MAM Preemie Schnuller für Frühgeborene, MAM Preemie narresut til tidligt fødte børn, Πιπίλα MAM Preemie για πρόωρα βρέφοι, Chupete MAM Preemie para bebés prematuros, MAM Preemie - keskos tutti, Sucette MAM Preemie pour bébés prématurés, MAM Preemie ciuccio per neonati prematuri, MAM Preemie smokk for premature barn, MAM Preemie voor premature baby's, Smoczek dla wcześniaków MAM Preemie, Succhietto MAM Preemie per neonati premature, MAM Preemie napp for prematura barn, MAM Preemie 早产儿安抚奶嘴

G210x297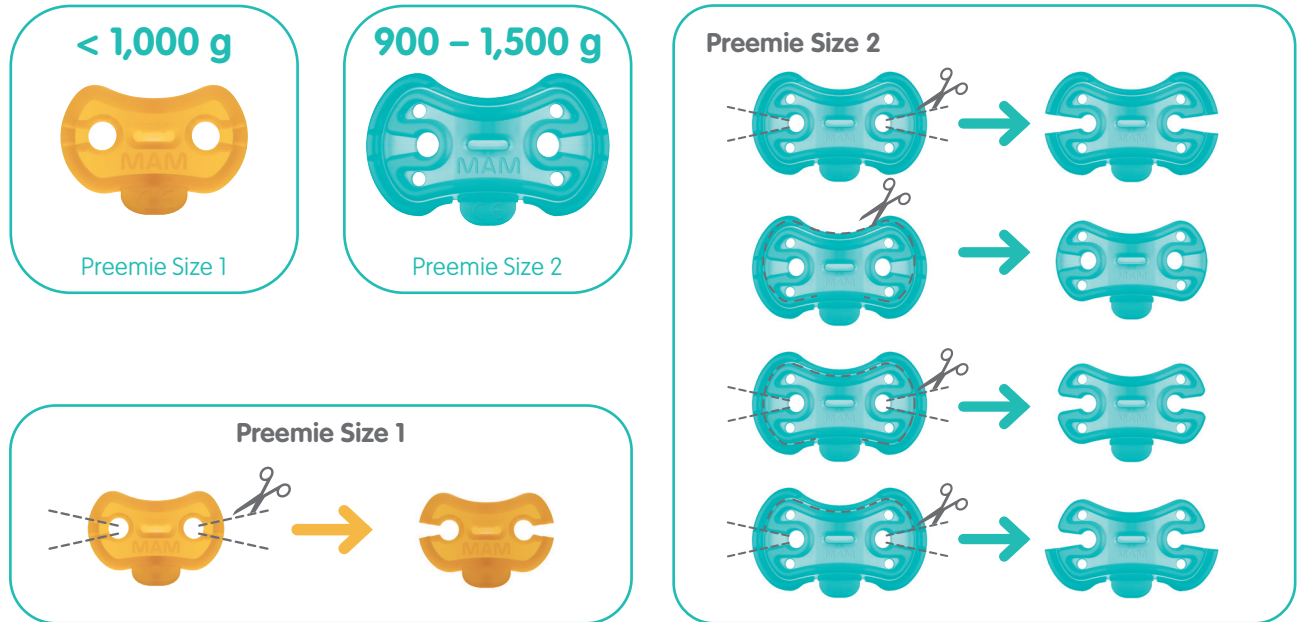

EN: Read WARNINGS and Instructions for use enclosed in every outer box. Product Training Sheet is available on mambaby.com/professionals. MAM Preemies are medical devices (CE) for premature babies and they are only suitable for hospital use. They are intended to compensate for the sucking and swallowing weaknesses in premature babies. MAM Preemies do not meet the safety standards which are applied to normal soothers namely EN 1400 and 16 CFR 1511. The product poses a choking hazard. Therefore, MAM Preemies must only be used under medical supervision. Parents must never be allowed to take the product home. Only use for one baby. Autoclavable from 121°C/250°F - 134°C/273°F, for 15 (min.) to 18 (max.) minutes (max. 20 times).

Follow instructions for use and on how to clean. Please give us your feedback on our product (product improvements, complaints, experience, etc.) For complaints, please return the product to us and enclose the date of purchase, the LOT-Number (printed on packaging, if available), pictures and a detailed description of the issue. Contact us on mambaby.com

UK: Return address: MAM UK, Marvan Court, Block C, Ground Floor, 1 Waldegrave Road, Teddington, TW11 8LZ, United Kingdom.

Contact email: medical@mamuk.com Office contact phone: 020 8943 8880. For emergencies outside office hours, please phone: 07968 065451.

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