# Preemie Product Training Sheet

MAM Preemie soother/pacifier for premature babies, Chupeta MAM Preemie para bebês prematuros, MAM Preemie Schnuller für Frühgeborene, MAM Preemie narresut til for tidligt fødte børn, Πιπίλα MAM Preemie για πρόωρα βρέφη, Chupete MAM Preemie para bebês prematuros, MAM Preemie - keskos tutti, Sucette MAM Preemie pour bébés prématurés, MAM Preemie ciuccio per neonati prematuri, MAM Preemie smokk for premature barn, MAM Preemie voor premature baby's, Smoczek dla wcześniaków MAM Preemie, Succhietto MAM Preemie per neonati premature, MAM Preemie napp for prematura barn. MAM Preemie cumi koraszülötteknek

## PREEMIE SIZE 1 & 2

For babies weighing:  
<1.000g

Preemie size 1

For babies weighing:  
900g – 1.500g


Preemie size 2

## FITTING OPTIONS FOR PREEMIE SIZE 1

Custom fitting

## FITTING OPTIONS FOR PREEMIE SIZE 2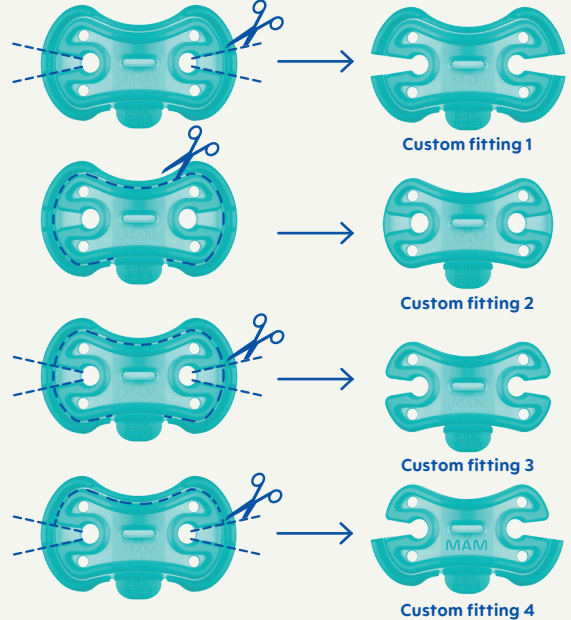

Custom fitting 1

Custom fitting 2

Custom fitting 3

Custom fitting 4

**EN:** Read WARNINGS and Instructions for use enclosed in every outer box. Product Training Sheet is available on [mambaby.com/professionals](http://mambaby.com/professionals). MAM Preemies are medical devices (CE/UKCA) for premature babies and they are only suitable for hospital use. They are intended to compensate for the sucking and swallowing weaknesses in premature babies. MAM Preemies do not meet the safety standards which are applied to normal soothers namely EN 1400 and 16 CFR 1511. The product poses a choking hazard. Therefore, MAM Preemies must only be used under medical supervision. Parents must never be allowed to take the product home. Only use for one baby. Autoclavable from 121°C/250°F - 134°C/273°F, for 15 (min.) to 18 (max.) minutes (max. 20 times).

Follow instructions for use and on how to clean. Please give us your feedback on our product (product improvements, complaints, experience, etc.) For complaints, please return the product to us and enclose the date of purchase, the LOT-Number (printed on packaging, if available), pictures and a detailed description of the issue. Contact us on [mambaby.com](http://mambaby.com)

UK: Return address: MAM UK, Marvan Court, Block C, Ground Floor, 1 Waldegrave Road, Teddington, TW11 8LZ, United Kingdom.  
Contact email: [medical@mamuk.com](mailto:medical@mamuk.com) Office contact phone: 020 8943 8880. For emergencies outside office hours, please phone: 07968 065451.

US: Email: [NICUcustomerservice@mambaby.com](mailto:NICUcustomerservice@mambaby.com). Phone number: 914-468-4048

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BPA/BPS  
FREE<sup>o</sup>

<sup>o</sup>BPA/BPS free:  
This MAM product is  
made from materials  
free of BPA & BPS.

Medical Device  
Class 1